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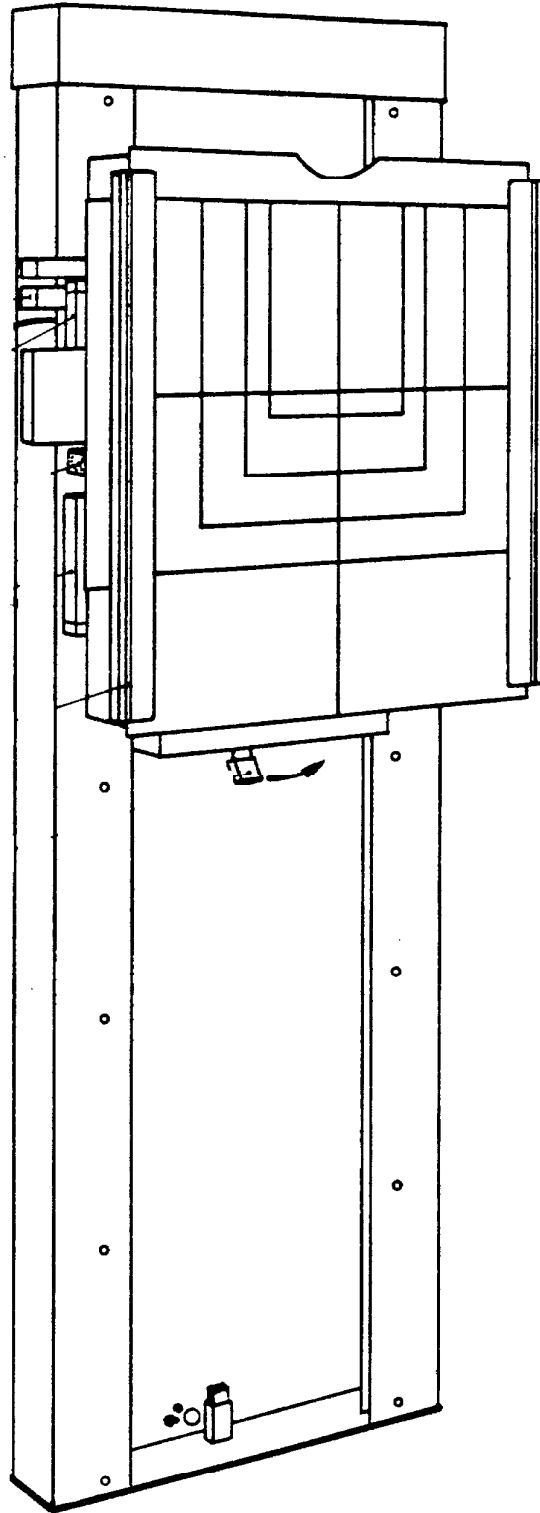
1. Description

The Bucky Stand BS 66 is available in 3 different versions:

"Vertical" Type 352350
 "Vertical-Turnable" Type 352300
 "Universal" Type 352001

All versions are available in free-standing or wall-mounted design. Operation side right or left, as desired. Adaption rails according DIN 650 at all stands for accessories such as head- or hip clamps, compression device etc.

Features of the different Versions

"Vertical": counterbalanced vertical movement of the carriage for bucky. Automatic locking in each position.

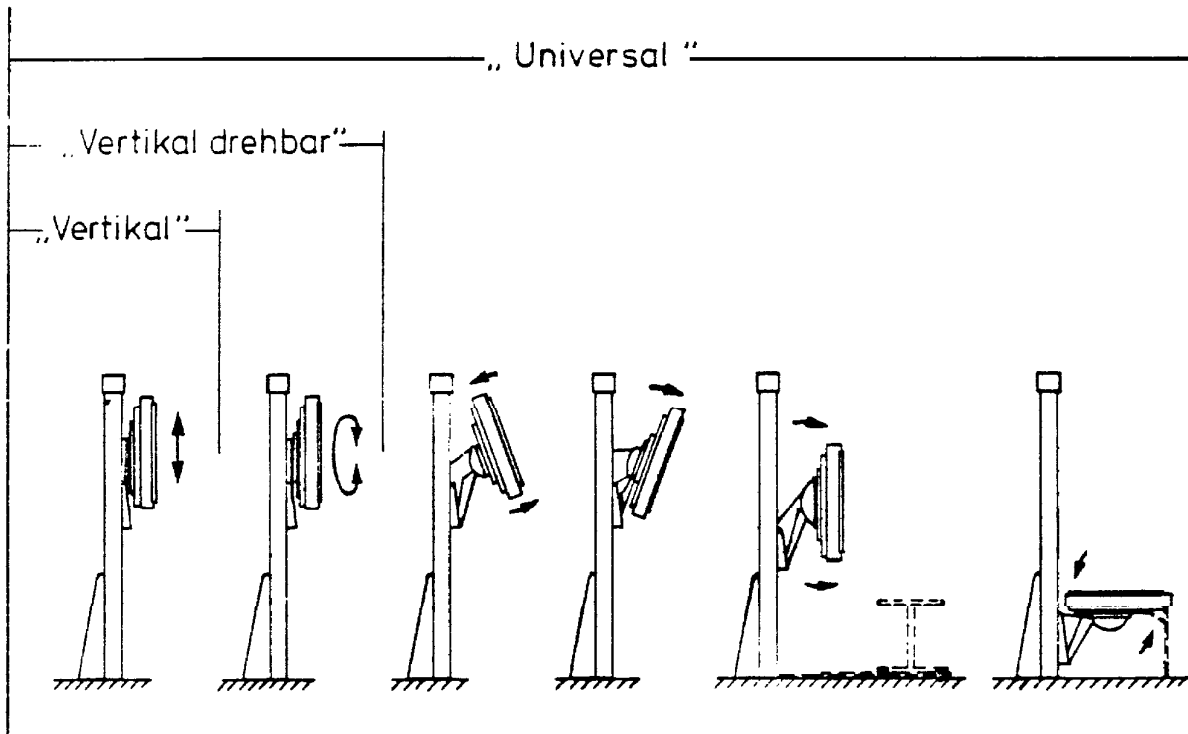
"Vertical-Turnable": Bucky is turnable in film plane by 180°. Can be locked in each intermediate position. Angle can be read from a scale.

"Universal": Angulation possibility of the Bucky up to horizontal position backwards as well as by 20° forward. Each intermediate position can be locked. Tilting angle can be read from the adapted goniometer. For exposures at the sitting patient, tilt the bucky by 20° forward and place it then in vertical position, in order to reach enough distance for the knees.

Movable patient stool (accessory part) with connection possibility and lockable telescopic guide, in order to center the patient to the stand.

Exposures at the patient sitting on the bucky, when using the leg assembly. For that purpose, the bucky must be in horizontal position. Height is 500 mm (adjustable back rest as accessory part available).

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2. Operation BS 66

(The position nos. refer to sheet no. 352 000 304, which shows the unit "Universal")

Unit "Vertical"

The bucky, mounted on a carriage, is counterbalanced movable and automatically locked. By pressing together both levers Pos. 1 in direction of the arrow, the lock is released and the carriage can be moved. The T-slotted rails Pos. 6 are for the adaption of accessory parts such as compression device, head clamps or back rest.

Unit "Vertical-Turnable"

Same as unit "Vertical" with the following addition: after having loosened the brake lever Pos. 2, the bucky can be rotated by max. 180°.

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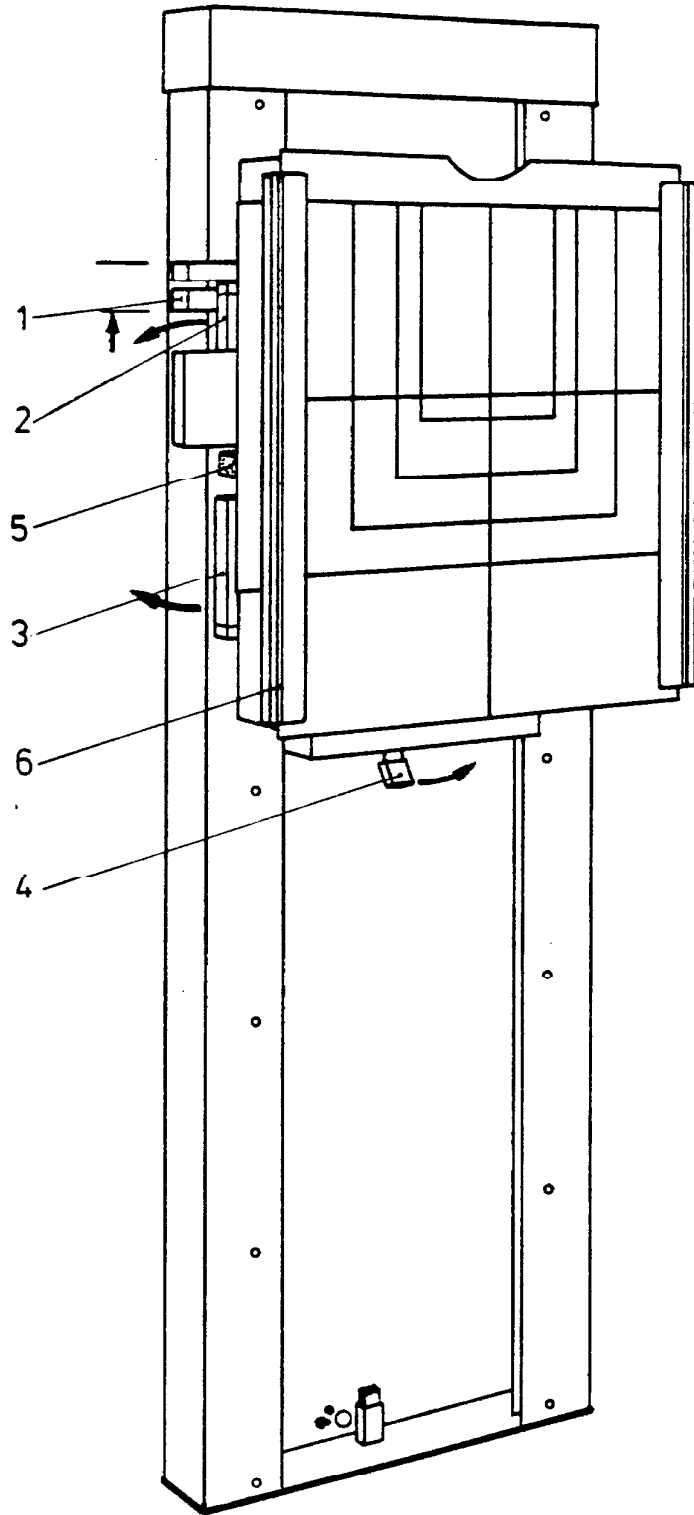
Unit "Universal"

Same as unit "Vertical" with the following addition: bucky can be rotated by 180° after having released brake Pos. 2. Can be locked in each intermediate position. The angle can be read from a scale.

After having released brake lever Pos. 3, the bucky can be placed in horizontal position (bucky must be rotated by 180° first). Any intermediate position can be locked. Angle can be read from the goniometer Pos. 5.

After having released the brake lever Pos. 4 in direction of the arrow, bucky can be angulated forward by 20° max. Angle can be read from the goniometer Pos. 5.

For exposures at patients sitting in front of the bucky, it must be angulated and then placed vertical, in order to have enough distance for the knees.

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3. Maintenance Schedule

The maintenance work, mentioned in Chapter 10 (sheet no. 3 352 010) must be done every 12 months by the service personnel. The steel ropes must be exchanged every 2 years for security purposes.

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Note:

All figures in these mounting in-
structions show the unit "Univer-
sal". All details which refer to
mounting are the same at all units.

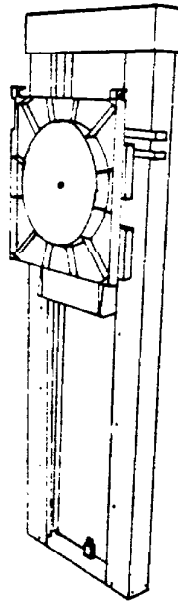
Fig.2

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7.2 Mounting

Put the stand on a wooden base.
Unscrew head part, Fig. 3 Pos. 2
and remove it in upward direction.
Insert the loop of the rope Pos.
8 into the rope suspension at
the carriage, Fig. 6 Pos. 15.
Put on the disc Pos. 10 and the
safety disc Pos. 9 according
Fig. 4.

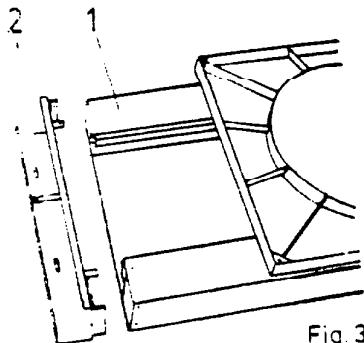
Fig.3

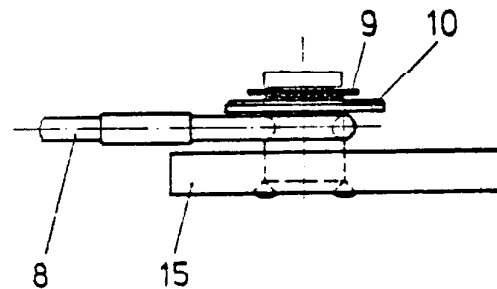
Fig.4

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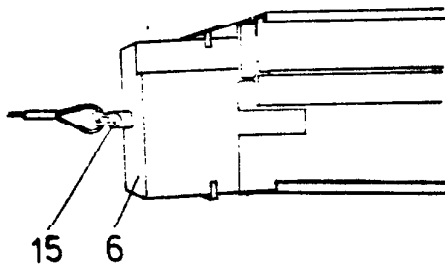
Fig.5

Fasten the rope at the weight pos. 6 by means of a link Fig. 5 Pos. 15. Put the weight into the shaft of the stand. Please care that the groove of the weight is at that side of the shaft, where the heads of the screws project. Insert the weight with the aid of a stick or something similar up to the middle of the stand. The same procedures must be done at the other side.

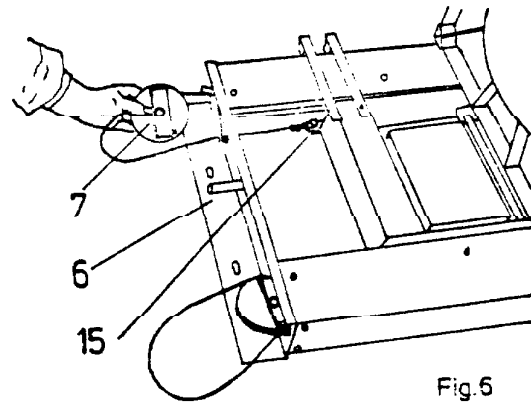
Fig.6

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Fasten head part Fig. 6 Pos. 2 again. Draw the ropes through the slots according Fig. 6. Put on the return pulleys Pos. 7 as shown in Fig. 6. Release brake of vertical carriage and move it downwards until the ropes are tightened.

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Put up the unit. Before fastening, lead the earth wire from behind through the hole at the earth clamp Fig. 7 Pos. 16. Fasten the unit vertically with safety dowels Fig. 1 Pos. 12 above and Pos. 11 below. Please consider diagrams 352 000 101 or 352 000 102 or 352 000 103 as well as mounting instructions for Liebig safety dowels.

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Attention: When the stand is vertical, the brake of the vertical carriage must only be released when the carriage is loaded with the weights or pressed down by hand. If the brake is released when the carriage is unloaded, there is danger of injury because of the sudden upward travel of the carriage. Furthermore the brake can get damaged.

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7.3 Adaption of Bucky

With units "Vertical-Turnable" and "Universal" both lugs of the plate diaphragm (see Fig. 7) must show upwards before the adaption of the bucky. Fasten bucky frame and bucky (chin rest upwards).

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7.4 Adjustment of Counterbalance

Press upon the bucky. Release the brake. If the carriage moves upwards, put so many weights Fig. 1 Pos. 13 into the weight box Fig. 8 Pos. 17 until the carriage is counterbalanced. It is recommended to consider an upward travel of approx. 20 to 30 N in order to counterbalance a cassette and possible accessories.

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Note: Fig. 8 shows the unit "Universal" where the weight box is visible from the front after having tilted the bucky. At the other types, the weight box is at the same point. Inserting of the weights is effected from above or from askance.

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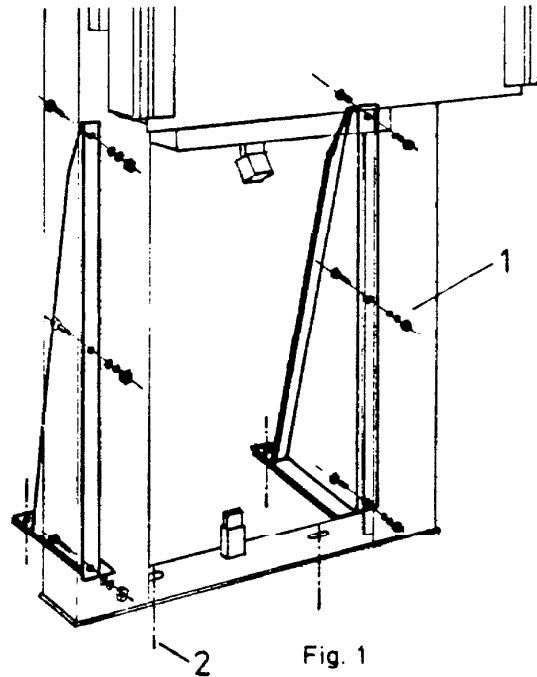
8. Mounting of BS 66 free-standing
(Special Accessories)

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Loosen 3 screws M5x12 each with disks, spring washers, and nuts. Use longer screws M5x16 and fasten supports to the stand Fig. 1.

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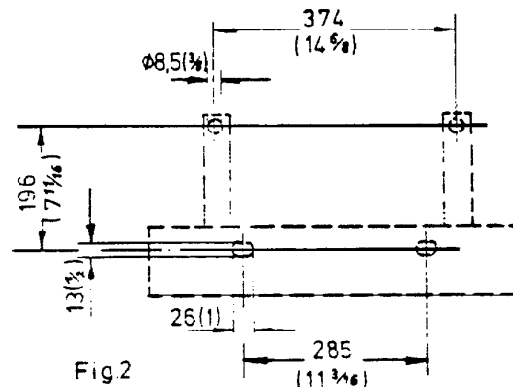
Fastening on concrete floor with 2 safety dowels at the stand and 1 safety dowel at each support. Use Liebig safety dowels B12/65N when the concrete floor has strength DIN 1045. Consider Liebig mounting instructions.

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If the floor is of another condition, the stand must be fastened in another manner. Diagram of holes Fig. 2, measures in mm (inch).

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10. Instructions for Maintenance and Repair

10.1 Visual Control and Maintenance

The ropes must be checked upon wear and rust every 12 months by visual control and feeling. Control can be effected by moving the carriage in lowest position. If wear at the ropes is noticed, they must be replaced immediately. After control, the ropes must be lubricated with non-acid Vaseline. Do not clean the ropes with dissolving material.

The steel ropes must be exchanged every 3 years.

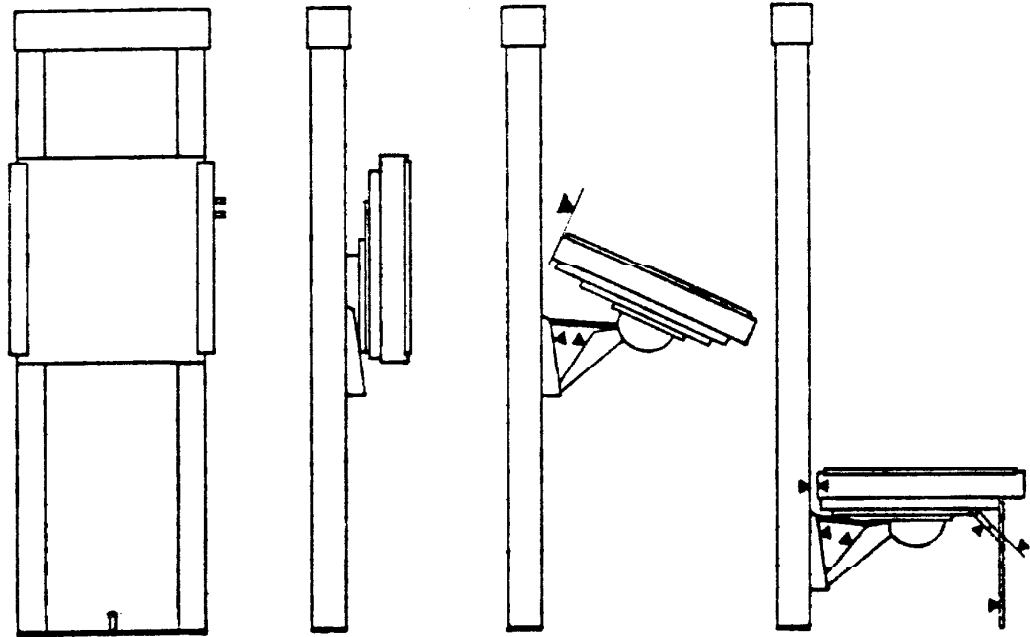
10.2 Replacement of the Rope

1. Tilt the stand
2. Loosen the head part
3. Move the carriage to the upper end of the stand
4. Remove the rope pulley
5. Unhinge the end of the rope at the carriage
6. Draw the weights to the upper end of the stand and unhinge the rope
7. Fasten the new rope at the weight
8. Move the weights to the middle of the shaft
9. Fix the rope at the carriage and insert the rope pulleys
10. Tighten the rope by moving the carriage
11. Screw on the head part
12. Put up the stand

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
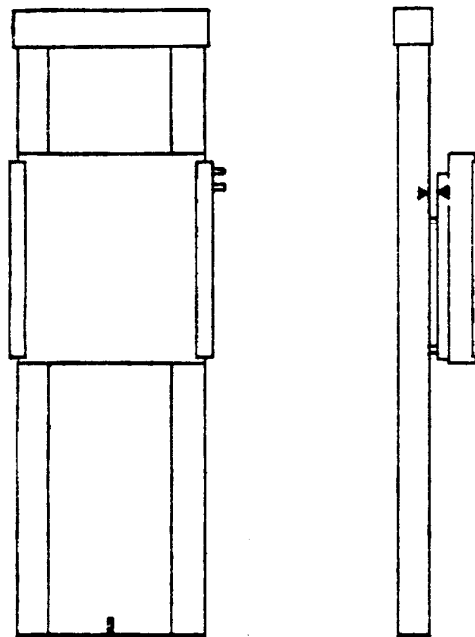
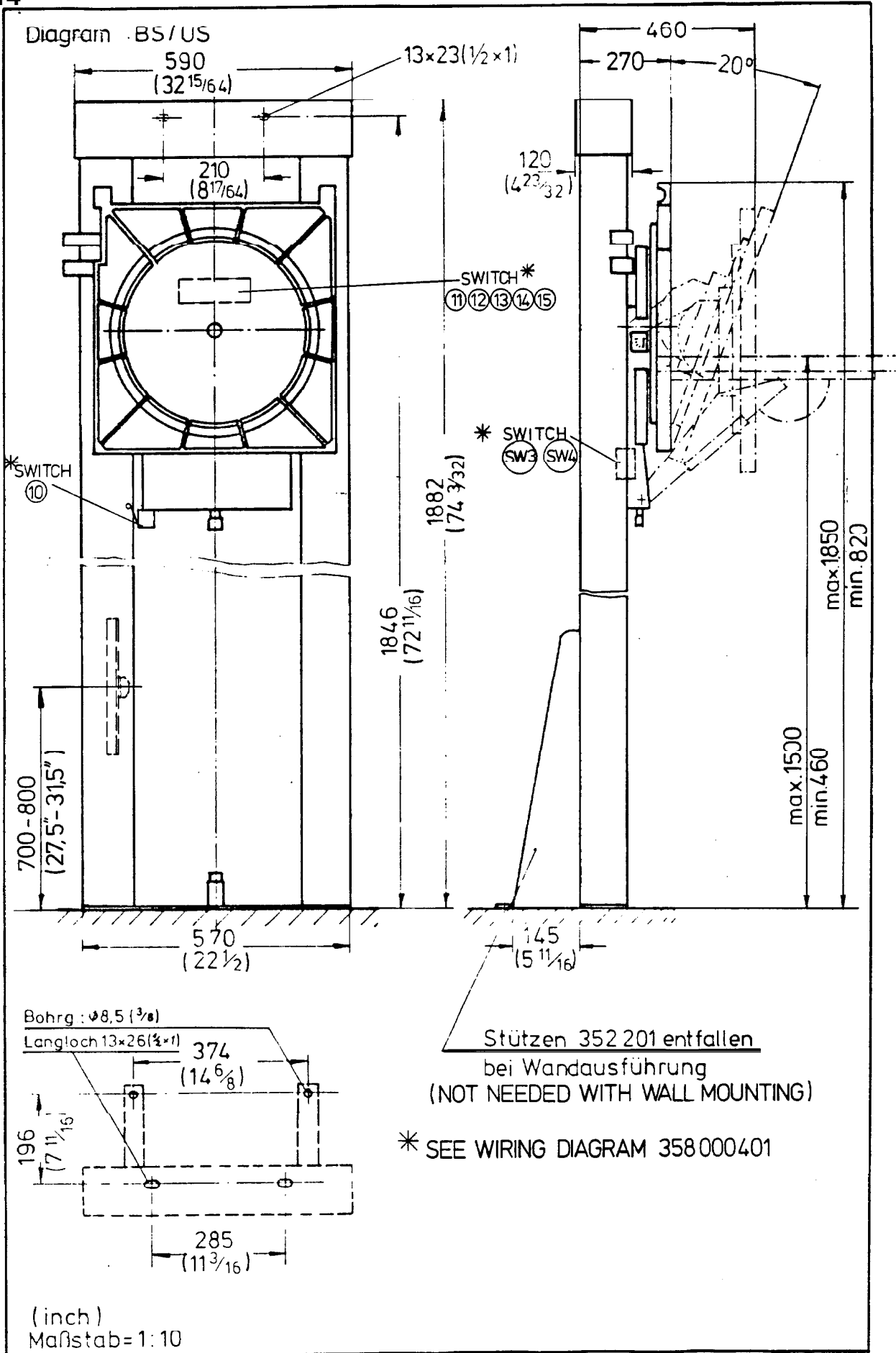
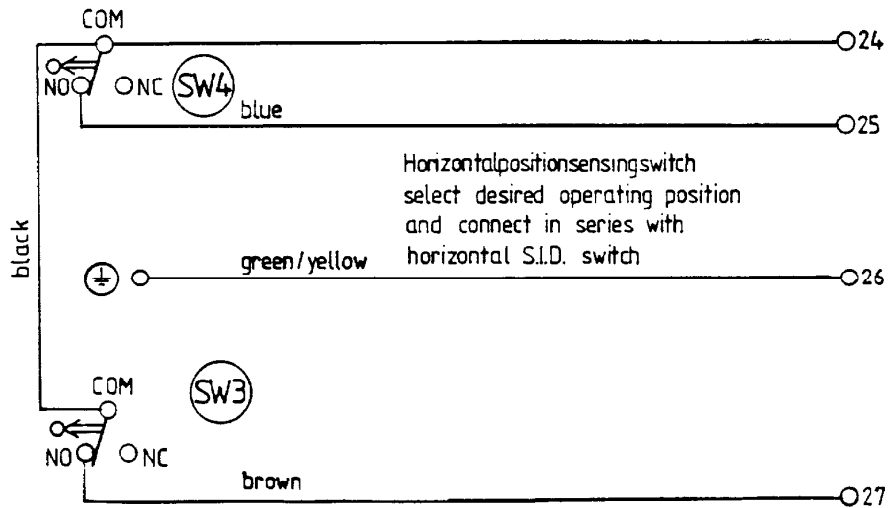
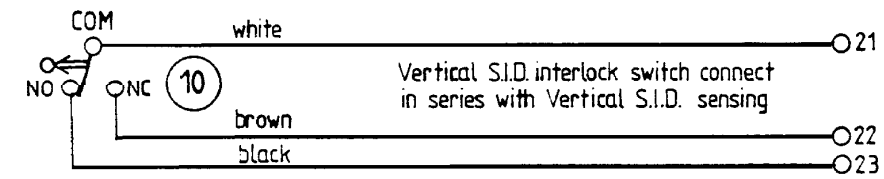
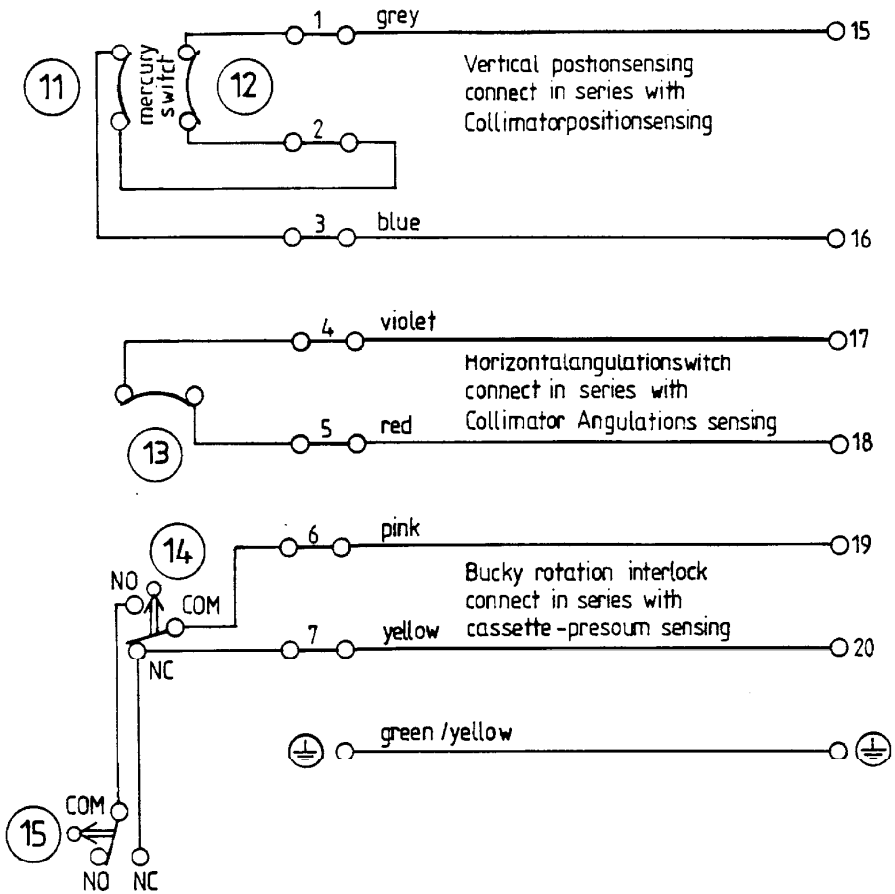
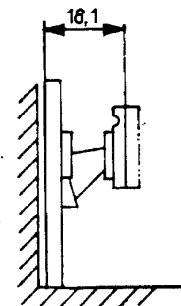
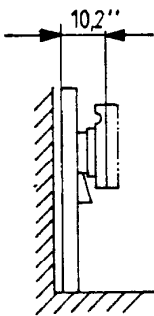
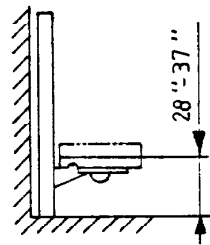
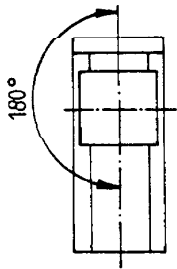
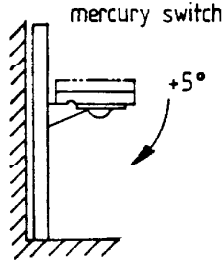
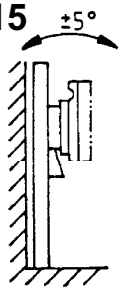
358 000 401
② 17.02.86 Bx

WIRING DIAGRAM FOR BUCKY STAND / TILTING

GE Version – INSTALLATION OF COUNTERWEIGHTS AND COUNTERWEIGHT CABLES

3-1 Counterweight Cable Installation (Not applicable to installation of new units shipped with counterweights factory installed)

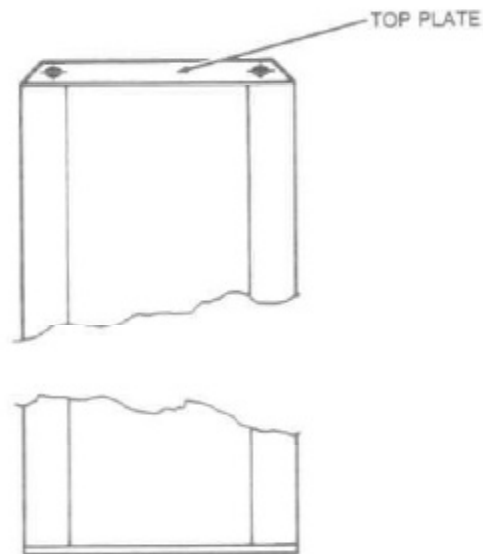
1. Remove the top plate (6 set screws), Illustration 3-1.

32940-D10

ILLUSTRATION 3-1

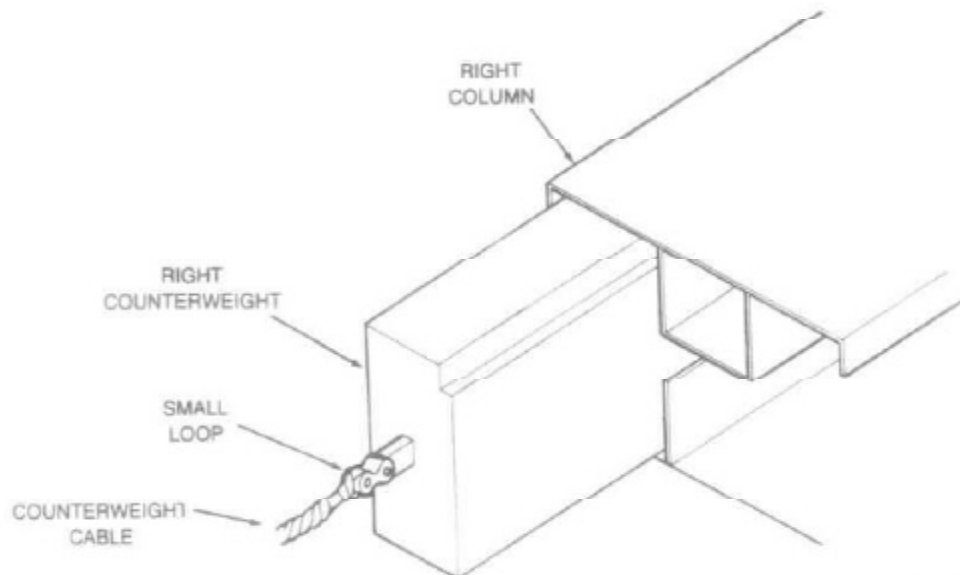2. At each end of the counterweight cable there is a loop. Use the smaller loop to connect the counterweight cables to counterweights (Illustration 3-2). Insert the counterweights inside the left and right columns of the stand. A broom handle can be used to push the counterweight 1/4 of the way down into the column.


32941-D12

ILLUSTRATION 3-2

3. Remove the upper bracket for the vertical height interlock cable (Illustration 4-1), and remove the vertical height interlock cable.

CAUTION: Bearings are held in place by a wire and pin. As the bearings disengage or engage in the tracks, the wire may pull out of the pin. The pin may then fall inside the carriage. Use a piece of tape to hold pin in place.

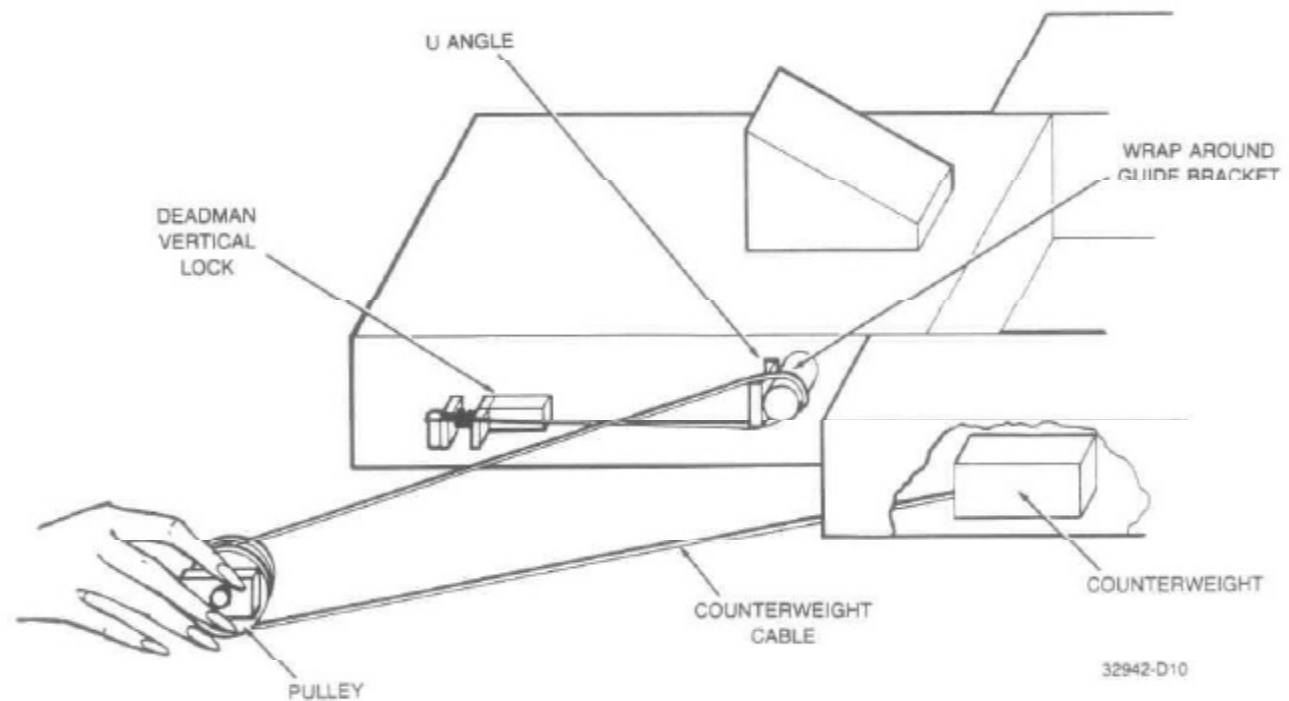4. Disengage the Vertical Lock by holding the two-brake lever (shown in Illustration 4-1), handles together and taping them with masking tape. Push the carriage beyond the upper end of the stand until the wrap around guide bracket is visible (Illustration 3-3).
5. Wipe all bearing rails with a rag moistened with a mixture of STP (or similar lubricant) and machine oil. Lubricate the counterweight cables by wiping with a cloth saturated with STP (or similar lubricant).
6. On the left and right sides of the unit, place the counterweight cable according to Illustration 3-3. The cable goes from the counterweight around the pulley, around the wrap around guide bracket, and place the (larger) loop at the end of the cable through the Lever on the Deadman Vertical Lock (Illustration 3-4).
7. On each counterweight cable (Illustration 3-3), insert the two joints of the U-Angle so that the counterweight cable will not slip off the wrap around guide bracket.
8. Push the carriage back into the stand by:
 - Releasing the deadman lock by pulling the counterweight cables taut.
 - And manually pushing the carriage back.

SIDE VIEW OF COUNTERWEIGHT CABLE ROUTING
ILLUSTRATION 3-3

PIN 46-179908P240

24914-P10


Pausch P/N 0353 0083

DEADMAN VERTICAL LOCK

ILLUSTRATION 3-4

3-2 Installation of Additional Counterweights Only For Units WITH Accessories (Head Clamps, Hip Clamps, Compression Device or Extending Handle)

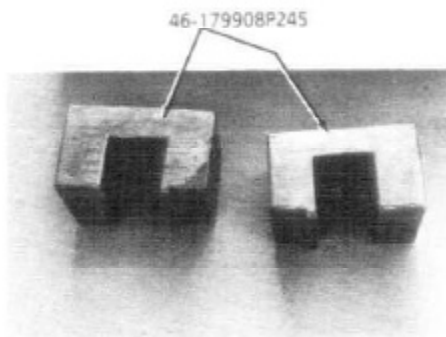
1. If accessories are used, then an additional weight must be placed upon the counterweights. (See Illustration 3-5)
2. Place the additional weights onto the counterweights per Illustration 3-6.

24916-P10

ADDITIONAL COUNTERWEIGHTS
FOR ACCESSORIES

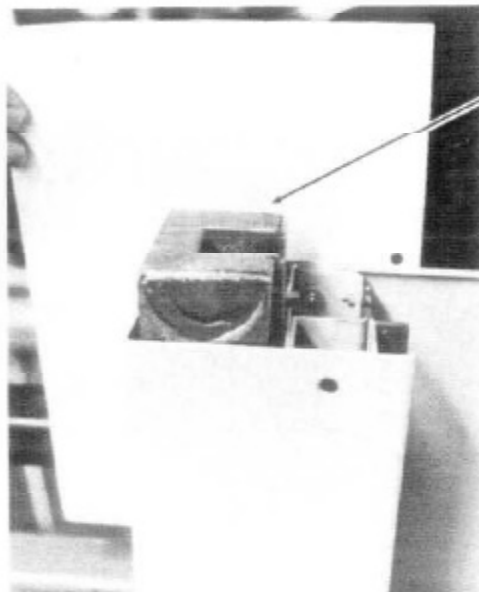ILLUSTRATION 3-5

ADDITIONAL
COUNTERWEIGHTS
FOR ACCESSORIES

24918-P10

ILLUSTRATION 3-6

Installation of Ballast Weight Only For Units WITHOUT Accessories

1. If accessories are not used, a ballast weight must be placed on the carriage (Illustration 3-7)


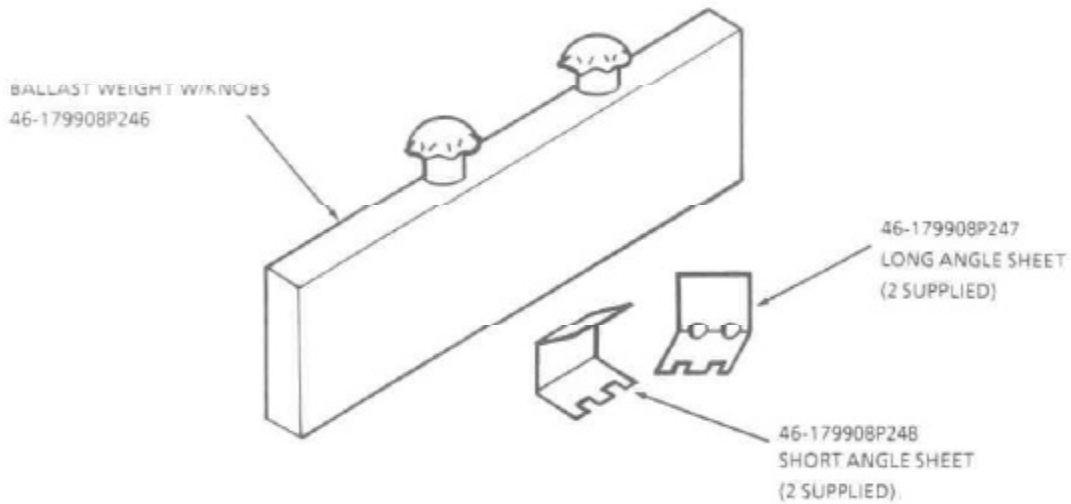
ILLUSTRATION 3-7

32943-D10


2. Loosen the screws on the top of the brake handle and loosen the screws on the trim cover which hides the holes for the brake lever (since the brake lever can be mounted on either side, there are holes on **both** sides. The side without the brake lever has a trim cover to hide these holes).

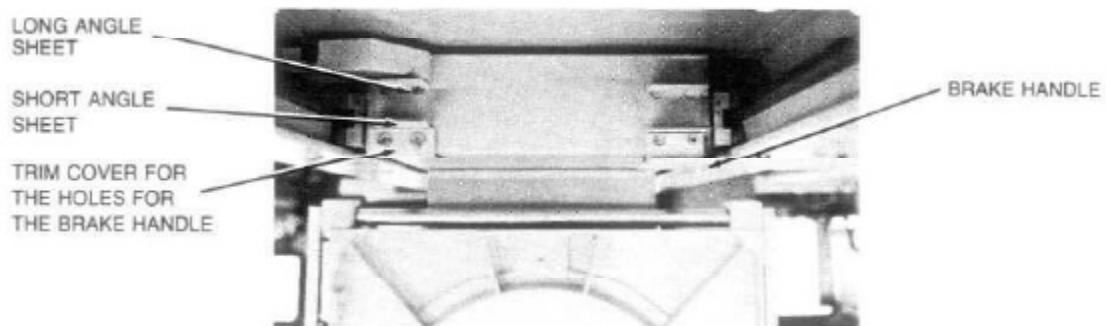CAUTION: The brake may not hold very well once the screws are loosened.


14919-P10

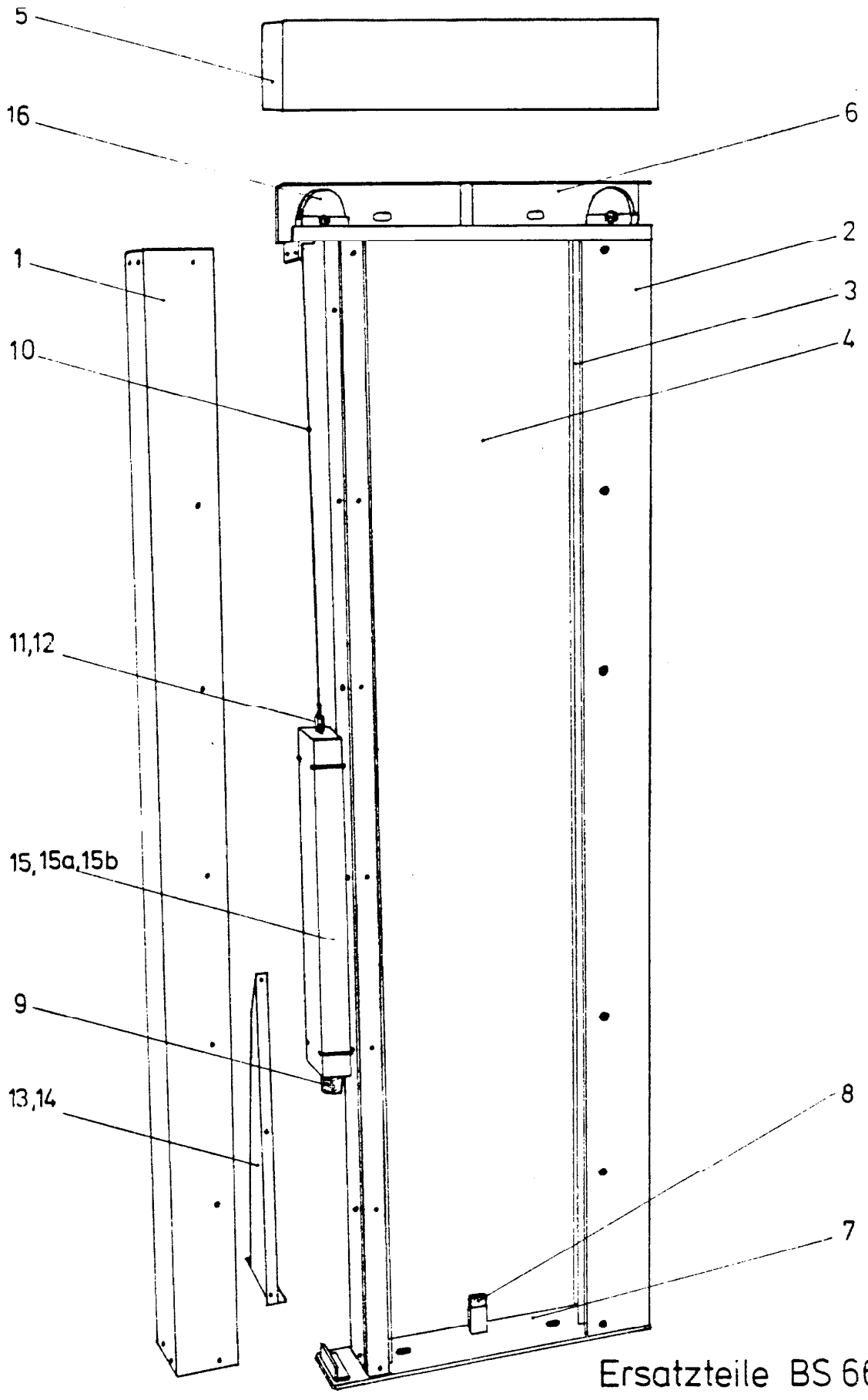TOP VIEW OF ANGLE SHEET MOUNTING (RIGHT-HAND STAND)
ILLUSTRATION 3-8


352000206
1 / 5
7.10.77

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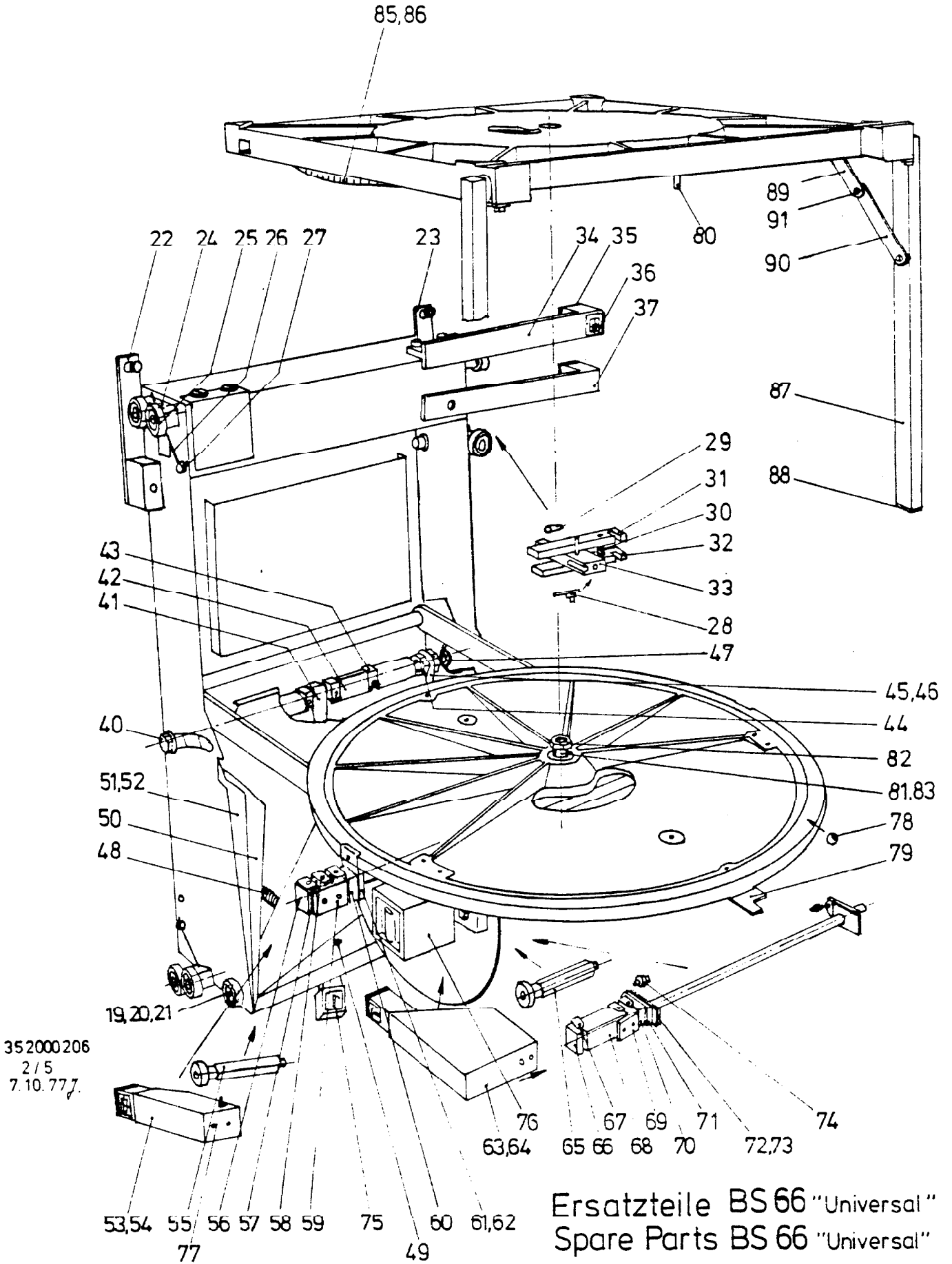
BS66 Vertical lock Modification

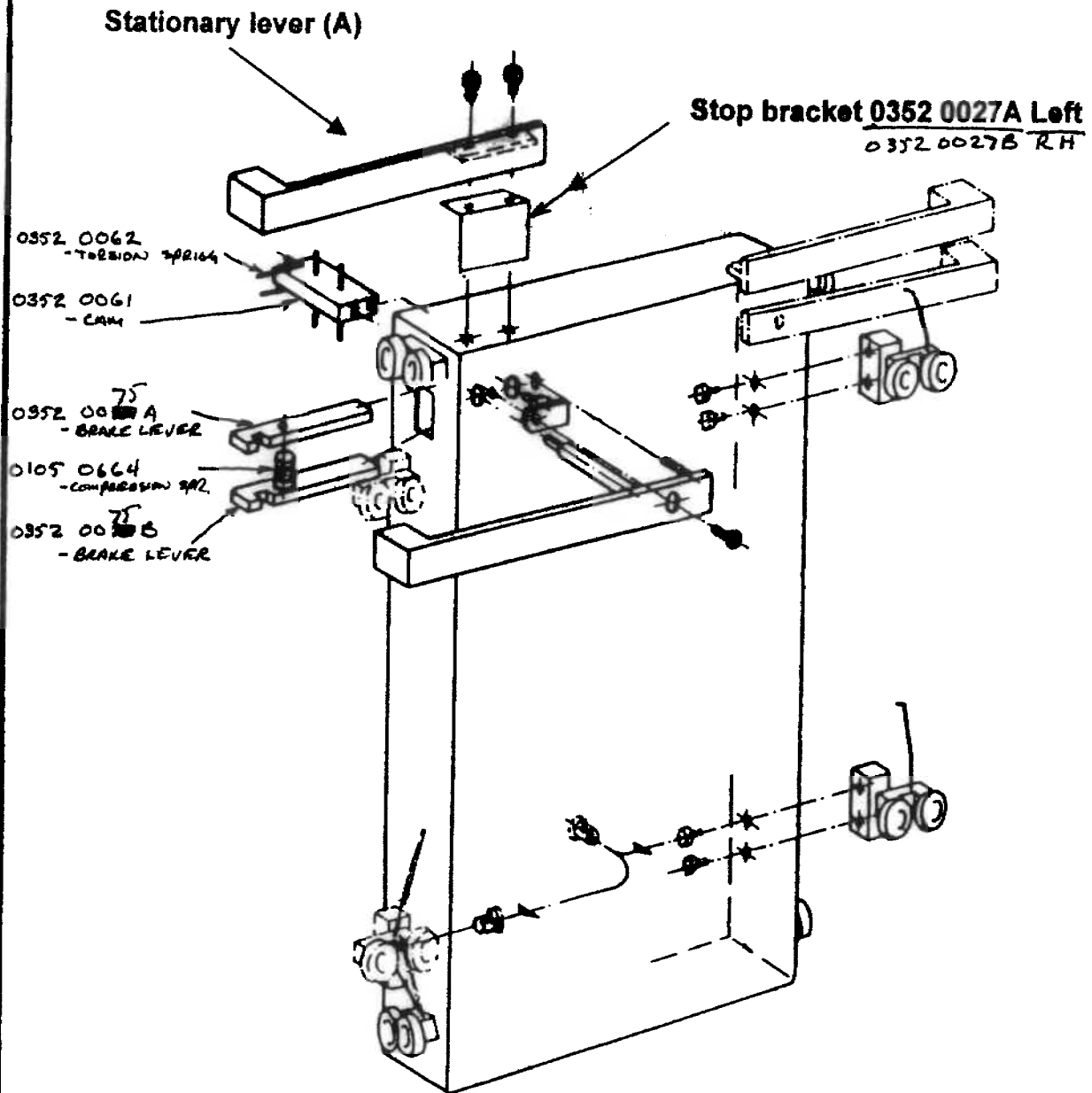
Procedure:

The purpose of the modification is to stop the active vertical lock release lever from dropping down to far. When this occurs the vertical locking plates rotate beyond top dead center and cause the lock to release.

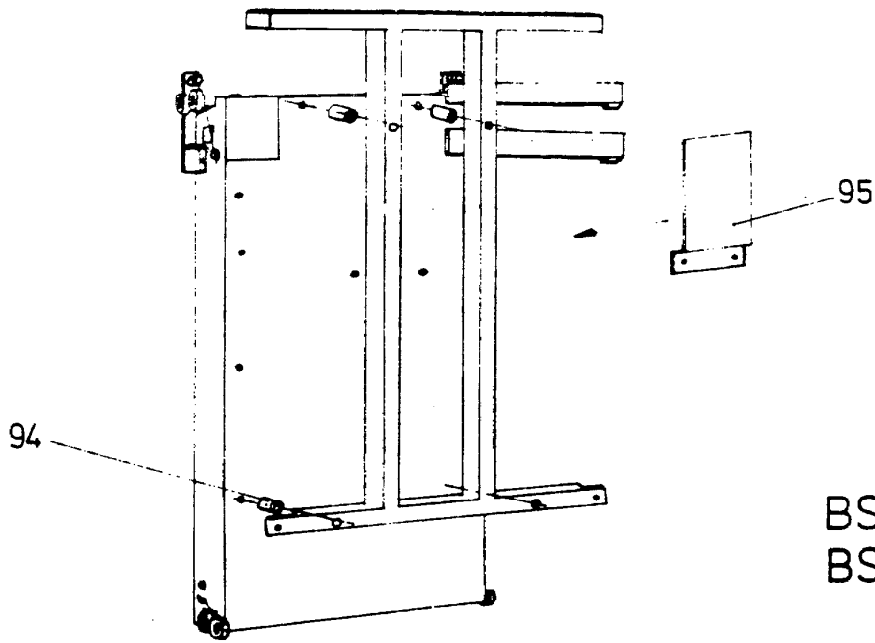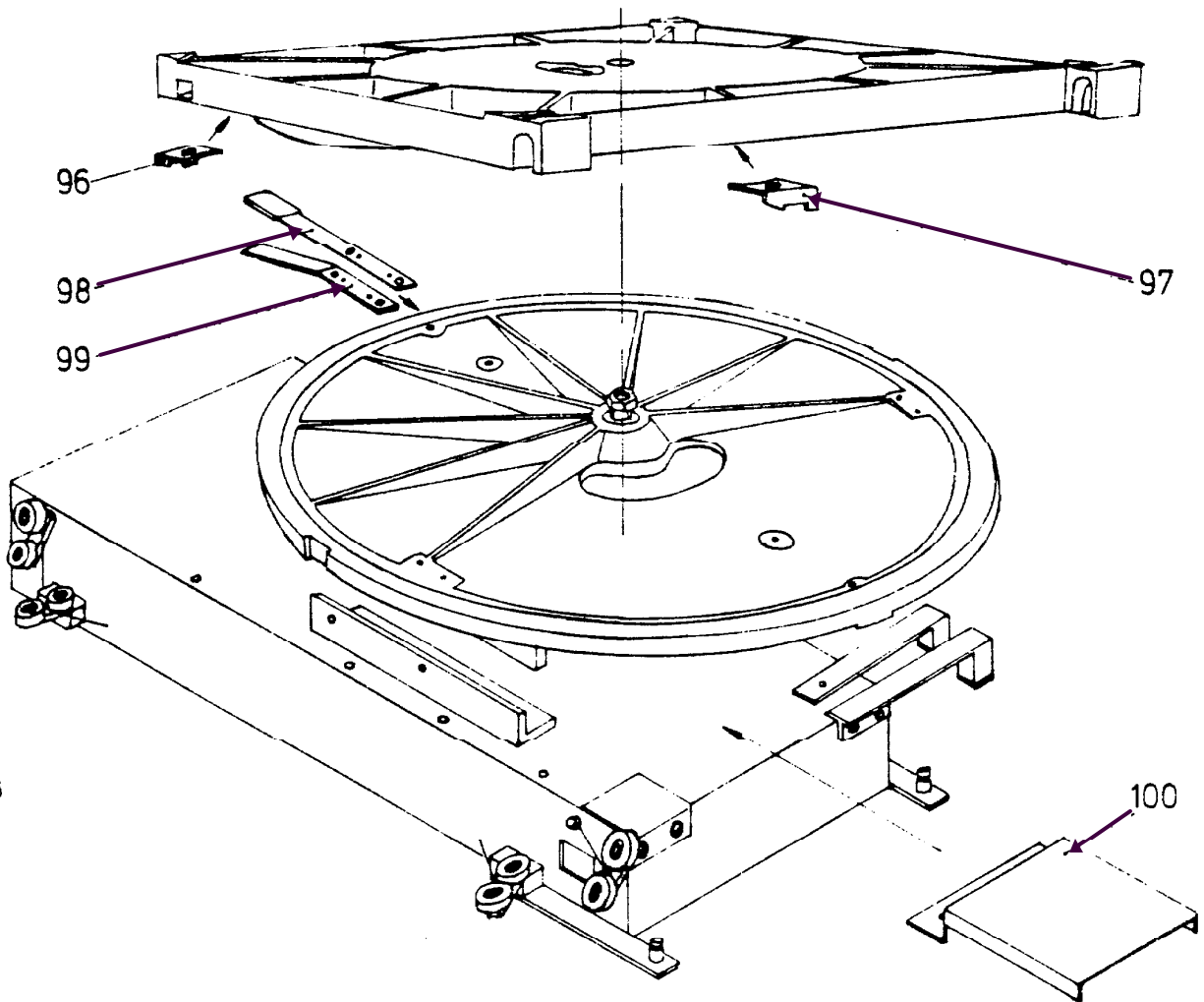To modify the Bucky stand lock remove the stationary lever (A) insert bracket 0352 0027a if the device is left-hand loading or 0353 0027B if it is right-hand loading. Reinstall the stationary lever.

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